



## Blood Glucose Monitoring System

### HS200 Blood Glucose Control Solution



- Blood glucose control solution for self-testing.
- Verify the accuracy of the Rossmax Blood Glucose Monitoring System and check the performance of the Rossmax HS200 meter, test strips, and your testing technique.
- Use within 90 days after opening.
- Only works with Rossmax HS200 glucose meter.
- IVD Directive 98/79/EC & EN ISO15197:2015 compliance.

#### Technical Specification

- Operation condition: Temp: 15°C~35°C, do not refrigerate
- Storage condition: Temp: 4°C~30°C, do not refrigerate
- Control Solution Level

Control Solution	mg/dL	mmol/L
Level 1	50	2.8
Level 2	130	7.2
Level 3	350	19.4

#### When To Perform a Control Solution Test

1. You get your meter for the first time before testing your blood test.
2. You open a new vial of test strips.
3. You want to check the meter and strips.
4. You suspect that the meter or test strips are not working properly.
5. Your blood glucose test results are not consistent with how you feel.
6. You think your test result is not accurate.
7. You dropped your meter.
8. Whenever your healthcare professional recommends it.