roSSmax

Model: **AM30**

EN Air Mattress Bubble Type

ES Colchón de burbujas

DE Bubble-Typ Matratze

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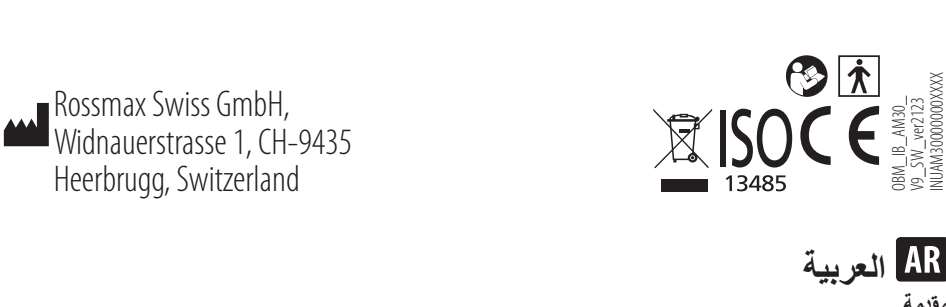
www.rossmax.com

Warranty Card
<p>This instrument is covered by a 1 year guarantee from the date of purchase, mattress and accessories are not included. For products sold within the European Union, rossmax acknowledges all consumer rights granted under the EU Directive 2019/771 and the respective national laws within the EU. The guarantee is valid only on presentation of the warranty card completed or stamped by the seller/dealer confirming date of purchase or the receipt. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or non-compliance with the instruction manual. Please contact your local seller/dealer or buying source or www.rossmax.com.</p>
<p>Customer Name: _____</p>
<p>Address: _____</p>
<p>Telephone: _____</p>
<p>E-mail Address: _____</p>
<p>Product Information</p>
<p>Date of purchase: _____</p>
<p>Store where purchased: _____</p>

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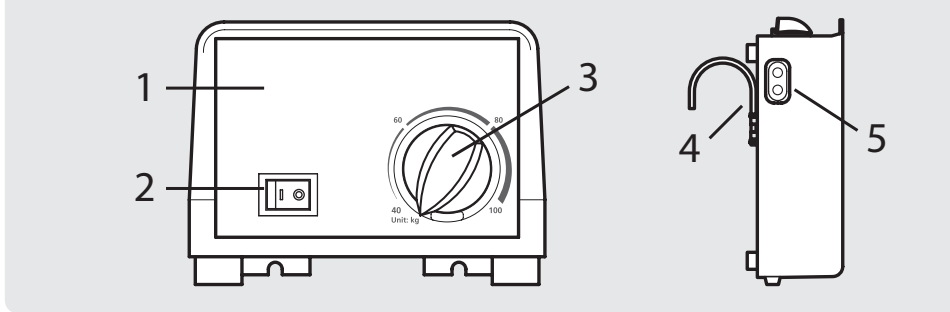
EN English

Introduction

Thank you for purchasing Rossmax bubble air mattress. With proper care and use, your air mattress will provide you with many years of reliable treatments. We encourage you to thoroughly read this guide-book to learn about setting up and the features of your air mattress. The mattress relieves pressure by sequentially deflating and inflating alternate air cells on approximately 6 minutes timed intervals. It is intended to help and reduce the incidence of pressure ulcers while optimizing patient comfort, long term care of patients suffering from pressure ulcers.

ⓘ Attention: Consult the accompanying documents. Please read this manual carefully before use.

ⓘ Please be sure to keep this manual.

Product caution

- Do not smoke on or near the pump.
- Keep the pump away from heated surfaces.
- Explosion risk if used in the presence of flammable anesthetics.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, please contact Rossmax service if necessary.
- Replace fuse as marked: T1A 250V.
- Do not use with patients who have a spinal cord injury.
- Close supervision is necessary when this product is used on or near children.
- Use this product only for its intended use as described in this manual. Do not use other mattress not recommended by manufacturer.
- Do not modify this product without authorization of the manufacturer.
- If you suspect that you may have had or are having an allergic reaction, please consult your physician immediately.
- Do not service or maintain device while in use.

How to use

- Place the mattress on the bed frame with the hose end at the foot section of the bed frame, Secure the mattress firmly by fixing the flags to the bed frame if available.
- Using the integrated hangers, securely hang the pump on the bed end at the foot end or place on a smooth flat surface.
- Connect the air hoses from the mattress to the pump unit.
Note: Check and ensure the air hoses are not kinked or tucked under mattress.
- Plug the pump into a wall electrical outlet. Be sure the power cord is safely away from possible hazards.
Note: After installation, the extra length of power cord, if any, should be neatly arranged to avoid any tripping accidents.
- Turn on the power switch on the control panel of the pump. The pump will begin to inflate the mat-tress.
Note: The pump should only be used with mattress recommended by manufacturer. Do not use it for any other purpose.
- While inflation, adjust the pressure setting based upon patient comfort level by turning the pressure adjustment knob clockwise to increase firmness.
Note: Every time the mattress is set up for use, it is recommended that the pressure first to be set to max., and then adjust air mattress weight levels to the desired softness after set up has been completed.

Clean and disinfection

- Wipe the pump unit with a damp cloth and mild detergent. If other detergent is used, choose one that will have no chemical effects on the surface of the plastic case of the pump unit.
- Wipe down the mattress with warm water containing a mild detergent and air dried thoroughly before use.
Note:
 - Do not use phenolic based product for cleaning.
 - After cleaning, dry the mattress without direct exposure sunlight.

Troubleshooting

Symptoms	Check point/Correction
Power is not ON	• Check if the plug is connected to mains. <ul style="list-style-type: none">Check for a blown fuse.
Patient is bottoming out	• Pressure setting might be inadequate for the patient, adjust higher pressure and wait for a few minutes for best comfort.
Mattress form is loose	Check if the mattress is fixed to the bed frame by flags.
No air produced from some air outlets of the air tube connector	This is normal since there is alternating mode, air outlets take turns to produce air during their cycle time.

Note: If the pressure level is consistently low, check for any leakage (tubes or air hoses). If necessary, replace any damaged tube or hoses or contact Rossmax service if necessary.

Specification

Pump unit	
Dimension (L x W x H)	26.5 x 13.5 x 10 cm
Weight	3 lbs (1.4 kg)
AC power source	AC 230V/50Hz or AC 220V/60Hz or AC 110V/60Hz
Cycle time	6 mins
Mattress	
Dimension (L x W x H)	79" x 35" x 3" (200 x 90 x 7 cm)
Extension flags	20"(50 cm) of both sides
Weight	5 lbs (2.3 kg)
Weight Capacity	220 lbs (100 kg)
Operating Condition	10°C to 40°C (50°F to 104°F),10% to 75% RH; 700~1060hPa
Storage & Transportation Condition	-18°C to 43°C(0°F to 110°F),10% to 95% RH
ⓘ	Type BF applied parts
IP Classification	IP21: Protection against harmful ingress of water and particulate matter

Electromagnetic Compatibility Information

1. This device needs to be installed and put into service in accordance with the information provided in the user manual.

2. WARNING: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the AM30, including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result. If higher IMMUNITY TEST LEVELS than those specified in Table 9 are used, the minimum separation distance may be lowered. Lower minimum separation distances shall be calculated using the equation specified in 8.10.

EN Name/Function of each parts

- Control Panel
- ON/OFF Switch
- Pressure Adjust Knob
- Hanger
- Air Hose Port

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Manufacturer's declaration-electromagnetic immunity		
The AM30 is intended for use in the electromagnetic environment specified below. The customer or the user of the AM30 should assure that is used in such an environment.		
Immunity test	IEC 60601 test level	Compliance level
	3 Vrms: 0,15 MHz – 80 MHz <p>6 Vrms: in ISM and amateur radio bands between 0,15 MHz and 80 MHz</p>	Electromagnetic environment-guidance
Conducted RF IEC 61000-4-6	3 Vrms: 0,15 MHz – 80 MHz <p>6 Vrms: in ISM and amateur radio bands between 0,15 MHz and 80 MHz</p>	Portable and mobile RF communications equipment should be used no closer to any part of the AM30 including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: d = 1,2 √P, d = 1,2 √P 80MHz to 800 MHz, d = 2,3 √P 800MHz to 2,7 GHz
	80 % AM at 1 kHz	80 % AM at 1 kHz

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Radiated RF IEC 61000-4-3	10 V/m <p>80 MHz – 2,7 GHz</p> 80 % AM at 1 kHz	10 V/m <p>80 MHz – 2,7 GHz</p> 80 % AM at 1 kHz	Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Interference may occur in the vicinity of equipment marked with the following symbol: <div style="font-size: 10px; font-family: monospace;"> ☸</div>
			NOTE1: At 80 MHz and 800 MHz, the higher frequency range applies. <p>NOTE2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>

More information on EMC compliance of the device can be obtained from Rossmax website: www.rossmax.com.

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EN WARNING: The symbol on this product means that it's an electronic product and following the European directive 2012/19/EU the electronic products have to be dispose on your local recycling centre for safe treatment.

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