EN English
Introduction
The chamber is intended to be used with metered dose inhaler (MDI) to deliver aerosol medication to your lungs as prescribed by supervision of a licensed physician and/or a respiratory therapist. The intended environments for use include the home, hospitals and clinics. Only for single patient use. Recommended patient population: AS175 with small facemask (0 - 1.5 years); AS175 with medium facemask (1 - 5 years) ; AS175 with large facemask (5 years+); AS175 with mouthpiece (5 years+).

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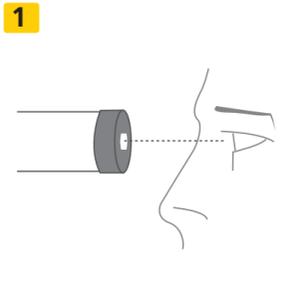
How to give an inhalation
Before use, carefully check the chamber. Remove any foreign objects from the chamber. Replace the chamber if any defect is noticed.

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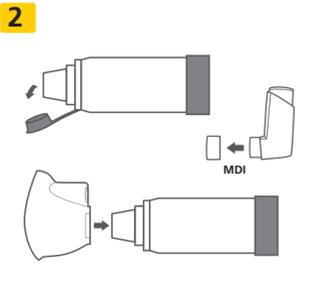
Remove cap from both MDI and chamber. When using the mask, insert the mask securely into mouthpiece.

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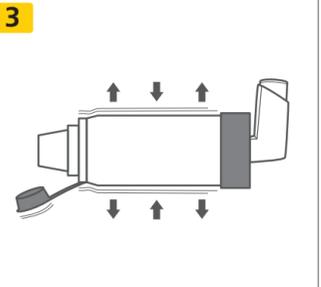
Insert the MDI into the back of chamber and shake the MDI and chamber.

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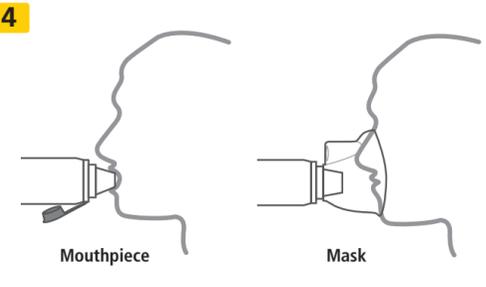
Put mouthpiece into mouth and close lips around it to ensure a tight seal. Apply the mask to mouth and nose and ensure a tight seal.

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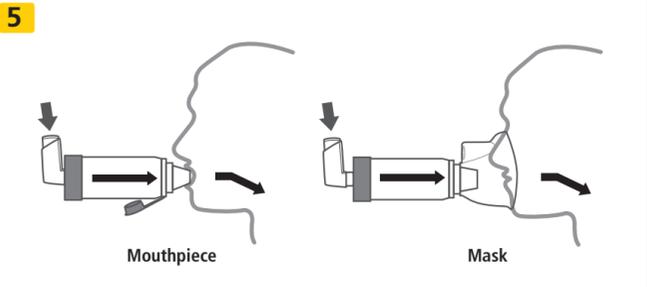
Breathe out gently. Press the MDI at the beginning of a slow inhalation. Then breathe in slowly and deeply through the mouth until a breath has been taken. Hold the breath for about 5-10 seconds, if possible. Otherwise, keep lips tight on the mouthpiece breathing about 2-3 times through the chamber after pressing the MDI.

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Breathe out gently. Press the MDI at the beginning of a slow inhalation, and then maintain seal for 5- 6 breaths after pressing the MDI.

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Follow instructions supplied with MDI on how long to wait before repeating steps 3-5.

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