

www.rossmax.com

How to record your peak expiratory flow rate

Rossmax Swiss GmbH, Tramstrasse 16, CH-9442 Berneck, Switzerland

08N_0P_Pfseie_0_SW_ver1 IN000X0000000000XX This meter was designed to make following your asthma treatment plan easy. Your doctor will set the color zone indicators to your personalized zones based on your personal best peak expiratory flow rate value (the best peak flow value you can achieve).

> **ISO CE** 9001/13485 0120

Note: Do not change the position of the color zone indicators yourself. If this should occur inadvertently, contact your doctor or special-

ist nurse for advice. Reading should be taken twice a day, first in the morning and at bed time, or directed by your physician. When you blow into your peak flow meter, the indicator will rise to your peak flow value. Your peak flow measurement will fall into one of three zones.

- **Green** follow your regular medication plan, and go ahead with normal activities (80-100% of best reading).
- Yellow caution, your asthma might be worsening. Follow your medication instructions to get back to your green zone (50-80% of best reading).
- **Red** Medical alert, get medical advice and attention immediately (< 50% of best reading).

DATE	SUN	MON	TUE	WED	THU	FRI	SAT	SUN	MON	TUE	WED	THU	FRI	SAT	SUN	MON	TUE	WED	тни	FRI	SAT	SUN	MON	TUE	WED	THU	FRI	SAT
DAY																												
🔆 midday	AM 🔆 PM	AM 🔅 PM	AM 🔆 PM	AM 🔆 PM	AM 🔆 PM	AM 🔆 PM	AM 🔆 PM	AM 🔆 PM	AM 🔆 PM	AM 🔅 PM	AM 🔆 PM	AM 🛒 PM	AM 🔆 PM	AM 🔆 PM	AM 🔆 PM	AM 🔆 PM												
800																												
700																												
600																												
500																												
400																												
300																												
200			Image: state																									
100																												
100																												
60																												