Model: **HP3040A**

**HP4060A**

- EN** Heating Pad
- DE** Heizkissen
- ES** Almohadilla Eléctrica
- FR** Tampon Chauffant
- RU** Электрическая Грелка
- NL** Verwarmingskussen
- AR** وسادة تدفئة
- FA** پد گرمايشی

**www.rossmax.com**

<b>Warranty Card</b>
<div><div><div><div><div><div><span></span></div></div></div><div><div><div><span></span></div></div><div><div><span></span></div></div></div><div><div><div><span></span></div></div><div><div><span></span></div></div></div><div><div><div><span></span></div></div><div><div><span></span></div></div></div></div></div></div> <div>This instrument is cover by a 2 year guarantee from the purchase date. The guarantee is valid only on presentation of the guarantee card completed by the dealer conforming date of purchase or the receipt. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or non-compliance with the instruction manual. Please contact your local seller/dealer or www.rossmax.com.</div>
<b>Customer Name:</b> _____
<b>Address:</b> _____
<b>Telephone:</b> _____ <b>E-mail address:</b> _____
<b>Product Information</b>
<b>Date of purchase:</b> _____
<b>Store where purchased:</b> _____

Manufacturer: Rossmax Swiss GmbH, Widnauerstrasse 1, CH-9435 Heerbrugg, Switzerland
<div><div><div><div><div><div><span></span></div></div></div><div><div><div><span></span></div></div><div><div><span></span></div></div></div><div><div><div><span></span></div></div><div><div><span></span></div></div></div><div><div><div><span></span></div></div><div><div><span></span></div></div></div></div></div></div>

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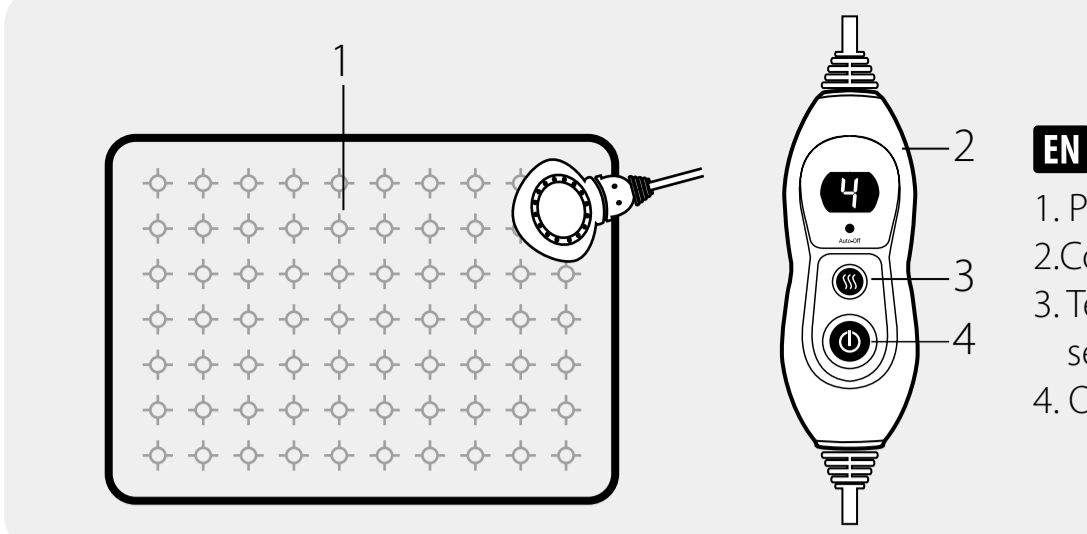
## EN English

**Introduction**

Thank you for your decision to purchase Rossmax heating pad. Please peruse this instruction manual prior to first start up and carefully keep this user instruction so as to have this information on hand whenever it is needed.

### Important information retain for future use

- This device is not intended for medical use in hospitals; for household use only.
- To avoid any risk of electric shock when using the pad, please observe the following:
  - Do not use if wet. If the pad has become damp or wet, completely dry the pad before use.
  - Only use the pad in dry rooms, not e.g. in the bathroom.
  - Check that the pad is undamaged each time before use, i.e. do not use it if there is visible damage to the hand control, switches, plug or the pad itself.
  - Do not insert pins or other sharp objects.
- To avoid overheating:
  - Do not leave the pad switched on unsupervised; switch it off when you leave the room.
  - Do not use the pad while folded.
  - Avoid creasing the pad.
  - Do not cover the controller! To ensure that the controller functions correctly, do not place it on or under the heating pad or cover it with other objects.
  - The cable should not be wound around the hand control.
- Do not use on a person incapable of movement, a baby, a person insensitive to heat, or animals. Do not let children play with the device.

- Ensure that children do not use this device unsupervised. This heating pad may not be used by children under 3 years old as they are unable to respond to overheating. Children above 3 and under 8 years of age must not use this device unless a parent or guardian has set the control to the lowest temperature setting 1.
- This device may be used by children aged 8 or over and by people who have reduced physical, sensory or mental abilities or inadequate experience and/or knowledge, provided they are supervised or have been instructed on how to use the device safely and have understood the resulting dangers.
- Excessive application at a high setting can result in skin burns.
- This heating pad may only be used in conjunction with the controller specified on the label.
- When the device is likely to be used for an extended period, i.e. by the user falling asleep, controls should be adjusted to a setting recommended for continuous use.
- Examine the device frequently for signs of wear or damage. If there are signs of wear or damage or the device has been misused, please contact the manufacturer or authorized agent prior to any further use.
- If the power cable is damaged, it must be replaced by the manufacturer, an authorized agent in order to avoid any hazards.
- Do not attempt to open or repair the device yourself. Special tools are required to re-assemble the device.
- Neither pierces the pad with, nor attaches safety pins or other spiky or sharp objects to the pad.
- This device emits very weak electrical and magnetic fields which may disturb the function of your cardiac pacemaker. Therefore, we recommend you to consult your physician and the manufacturer of your cardiac pacemaker before using this device.
- Allow the device to cool down before folding and storing it in a dry place. Do not place heavy objects on it to prevent buckling.
- Heat sources, such as hot water bottles, heating pads or similar, must not be placed on the pad.

**How to use**

- Plug the unit into the mains outlet.
- Press ⏻ and display the temperature at 1 setting. After a few minutes you will be able to feel the pad getting noticeably warmer. If you require a higher temperature setting, continuously press ⏻ and the displayed temperature circulates from 1 to 4 selections. Setting 1 is recommended for continuous use.
- The heating pad will switch off automatically and LED indicator will light up after approx. 90minutes of continuous use. To switch it back on again, Press ⏻ and then back to the desired heat setting.
- Unplug the unit from the mains outlet once you have finished using it.

**Cleaning and maintenance**

- Before cleaning the pad, unplug it from the mains outlet and let it cool down for at least ten minutes.
- The heating pad is equipped with a detachable cable, disconnect the plug and remove the cable from the heating pad.
- The heating pad may be washed according to the instruction on the laundry label.
- Store the unit in a clean and dry place, laid out flat with nothing covering or resting on top of it.

**⚠ WARNING**

- Do not allow controller to get wet.
- Do not use any detergents or solvents as they may damage the device.
- Never hold the controller under water. Otherwise moisture may enter and cause damage.
- Dry the pad completely before switching it on again.
- Cleaning and maintenance shall not be made by children without supervision.

Model:	HP3040A / HP4060A
Power Supply:	220-240 V AC 50 Hz
Heat output:	100 watts
Auto switch off:	approx. 90 mins
Operating condition:	Only use in dry rooms
Storage condition:	Laid out flat, clean and dry
Dimensions:	30 x 40 cm / 40 x 60 cm

**Explanation of symbols**

<span><span>📖</span></span> Read the instructions	<span><span>🚫</span></span> Do not bleach
<span><span>🚫</span></span> Do not insert pins	<span><span>🚫</span></span> Do not tumble dry
<span><span>🚫</span></span> Do not use folded or rucked	<span><span>🚫</span></span> Do not iron
<span><span>🚫</span></span> Not to be used by very young children (0- 3 year).	<span><span>🚫</span></span> Do not dry - clean
<span><span>🧺</span></span> 30°C gentle washing program	<span><span>⚠</span></span> <b>WARNING:</b> Warning of risks of injury or health hazards.
<span><span>🚫</span></span> Do not wash	<span><span>🚫</span></span> Do not use on pets

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